

Instructions for using custom-made insoles Secosol® and modifications for PPE footwear

Dear Customer,

you have today received the insoles/modifications for PPE shoes which have been specially prepared and tested for you. To ensure that you will be able to walk without difficulty please note the following instructions for their use and care:

- 1) The insoles are exclusively to wear with socks.
Initially your insoles/modified shoes should only be worn for periods of one hour at a time during the first 8-10 days, increasing the period of wear gradually day by day. The insoles should only be used in safety shoes they have been tested for.
- 2) Remove the insoles from the shoes regularly in order to air them. To clean them simply rub them with a damp cloth. Any dirt on shoes of this kind should not be treated with strong cleaning agents. Insoles and shoes which are damp should never be dried close to a central heating radiator or other heat source.
- 3) Safety shoes with a modified sole structure must only be used in temperatures of up to 60° C.
- 4) The support equipment provided has been produced especially in accordance with the Regulation (EU) 2016/425 of protective equipment (PPE) and meets the specifications of Annex I of EEC Directive 93/42. It is to be used only by the person, for whom it has been produced.
- 5) The Modifications of the PPE Shoes have been implemented by a notified testing body. You'll find the certified PPE Shoes with insole-system Secosol® in the download-area on www.secsosol.de
- 6) The distributing company indicates, that the insoles are according to Medical Device Regulation (EU) 2017/745 (MDR)



Example

The identification markings in the shoes indicate:

- Manufacturer and CE markings
- Type designation, article number, size
- Symbol of protective function, production
- The norm applied

For further instructions for use and care please note the instructions provided by the shoe manufacturer.